*No. 1 choice
of hospitals
& mothers*

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EN Instructions for use | RU Инструкция по применению
HU Használati utasítás | CS Návod k použití
TR Kullanım talimatları | PL Instrukcja obsługi

Thank you for choosing the Mini Electric Breastpump. Breastmilk is the natural choice for your baby, but there are reasons and situations where breastfeeding is not possible. For over 50 years, it has been Medela's conviction that these mothers and babies can be supported. In close cooperation with breastfeeding experts, Medela, one of the market leaders, has researched and developed breastpumps that perfectly meet breastfeeding mothers needs. For mothers who insist on the best for their babies and themselves. Because you care.

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PL
134–159

Table of Contents

1. Intended use/Population – Contraindication	5
2. Meaning of symbols	6
3. Important safety information	8
4. Product description	10
5. Cleaning	12
5.1 Before using for the first time and after every use	12
5.2 Before using for the first time and once a day	14
5.3 Cleaning the motor unit	15
6. Preparing for pumping	16
6.1 Mains operation	16
6.2 Battery operation	16
6.3 Assembling the pump set	18
7. Expression	20
7.1 Preparing for pumping	20
7.2 Pumping	20
8. Storing and thawing breastmilk	24
8.1 Storage	24
8.2 Freezing	24
8.3 Thawing	24
9. Feeding breastmilk	25
10. Troubleshooting	26
11. Warranty/disposal	27
12. Complete care during breastfeeding time	28
13. EMC/Technical description	160
14. Technical specifications	167

1. Intended use/Population – Contraindication

The intended use and indications of breastpumps

Breastpumps are intended to be used by lactating women in the hospital or home environment for expressing breastmilk. Breastpumps are indicated to alleviate the symptoms of engorgement by expressing milk. In case of mastitis breastpumps support the healing process by removing breastmilk from the affected breast. Breastpumps are indicated to relieve sore and cracked nipples and to bring out flat or inverted nipples. Also breastpumps enable mothers to provide breastmilk for babies who cannot feed directly at the breast for a variety of reasons such as latch-on problems, cleft palate or premature babies.

Intended patient population/user

Adolescent paediatric or adult lactating women. Many women find it convenient, or even necessary, to use a breastpump to express and store their breastmilk once they have returned to work, are travelling or are otherwise separated from their baby. Women are often delighted to learn that a breastpump can be used as a supplement to breastfeeding and that some pumps are designed to mimic the suckling of a nursing baby.

Contraindications

There are no known contraindications for the Mini Electric breastpump.

2. Meaning of symbols

6

The warning symbol identifies all instructions that are important for safety. Failure to observe these instructions can lead to injury or damage to the breastpump!

When used in conjunction with the following words, the warning symbol stands for:

Warning

Can lead to serious injury or death.

Caution

Can lead to minor injury.

Note

Can lead to material damage.

Info

Useful or important information that is not related to safety.

Symbols on the packaging

This symbol indicates that the material is part of a recovery/recycling process.

This symbol indicates a carton package.

This symbol indicates to keep the device away from sunlight.

This symbol indicates to handle the fragile device with care.

This symbol indicates the temperature limitation for operation, transport and storage.

This symbol indicates the humidity limitation for operation, transport and storage.

This symbol indicates the atmospheric pressure limitation for operation, transport and storage.

This symbol indicates to keep the device dry.

This symbol indicates that the packaging contains products intended to come in contact with food according to regulation 1935/2004.

This symbol indicates unique GSI Global Trade Item Numbers (GTIN)

This symbol indicates do not dispose the device together with unsorted municipal waste (for EU only).

This symbol indicates to follow instructions for use.

This symbol indicates the date of manufacture (four digits for the year and two digits for the month).

Symbols on the device

This symbol indicates to follow instructions for use.

-  This symbol indicates the compliance with the essential requirements of the Council Directive 93/42/EEC of 14 June 1993 concerning medical devices.
-  This symbol indicates the manufacturer.
-  This symbol indicates do not dispose the device together with unsorted municipal waste (for EU only).
-  This symbol indicates a type BF applied part.
-  This symbol indicates the positioning of battery cell.
- IP22** This symbol indicates the protection against ingress of solid foreign objects and against harmful effects due to the ingress of water.

Symbols on the mains adapter

-  This symbol indicates that the mains adapter is a class II device.
-  This symbol indicates the compliance with China Compulsory Certification.
-  This symbol indicates that the mains adapter is safety tested.
-  This symbol indicates that the mains adapter is for indoor use only.
-  This symbol indicates the compliance with Canada safety requirements.
-  This symbol indicates the compliance with USA and Canada safety requirements.
-  This symbol indicates the compliance with USA safety requirements.
-  This symbol indicates the mains adapter is fitted with thermal cut-out (number indicates cut-out temperature)
-  This symbol indicates polarity of d.c. power connector.
-  This symbol indicates isolating safety transformer.
-  This CE-mark indicates the compliance with the low voltage and electromagnetic compatibility directive.
-  This symbol indicates to not dispose the device together with unsorted municipal waste (for EU only). The solid bar symbol indicates that mains adapter is put on the market after 13 August 2005.
-  This symbol indicates the compliance with Japanese safety requirements.
-  This symbol indicates alternating current.
-  This symbol indicates direct current.
-  This symbol indicates the compliance with the essential requirements of the Council Directive 93/42/EEC of 14 June 1993 concerning medical devices.

3. Important safety information

8

Failure to follow these instructions/safety information can lead to danger from the device. Subject to technical changes.

Device

Danger of electrocution! Keep the device dry! Never immerse in water or other liquids!

The Mini Electric breastpump is not heat-resistant: keep away from radiators and open flames.

Do not expose the motor unit to direct sunlight.

Repairs must be performed only by an authorised service agency. Do not repair yourself! No modifications to the device are permitted.

Never use a damaged device. Replace damaged or worn parts.

The operating life of the Mini Electric breastpump is 150 hours. The service life is two years.

Electrical use

Separation from the mains is only assured through the disconnection of the mains adapter from the wall socket outlet.

Keep the mains adapter away from heated surfaces.

Do not reach for any electrical device if it has fallen into water. Unplug immediately.

The breastpump should never be left unattended when plugged into a power source.

Use

Use the Mini Electric breastpump only for its intended use as described in this manual.

Never operate an electrical device if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.

Do not use the Mini Electric breastpump while bathing or showering.

Do not use the Mini Electric breastpump while sleeping or overly drowsy.

This is a single user product. Use by more than one person may present a health risk.

Do not drive whilst hands-free pumping.

Supervision is necessary when the Mini Electric breastpump is used in the vicinity of children.

Contact your lactation consultant or physician if you experience problems or pain.

Portable and mobile radio frequency communications equipment can affect the Mini Electric breastpump.

Important:

- | Plastic bottles and component parts become brittle when frozen and may break when dropped.
- | Also, bottles and component parts may become damaged if mishandled, e.g. dropped, over-tightened, or knocked over.
- | Take appropriate care in handling bottles and components.
- | Do not use the breastmilk if bottles or components become damaged.

4. Product description

10

Spare parts and accessories available for order can be found on page 28.

1x PersonalFit™ Breastshield M (24mm) (for other breastshield sizes see Chapter 12, page 28)

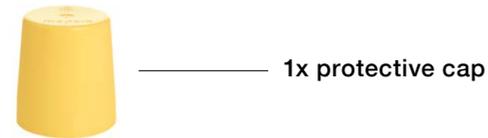
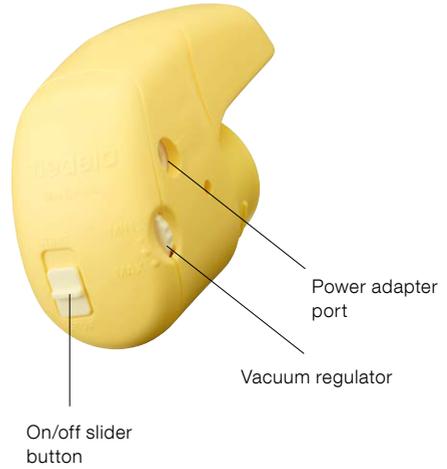
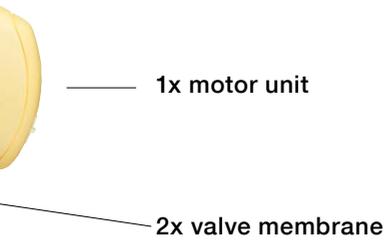
1x PersonalFit™ connector

1x valve head

1x bottle

1x bottle stand

5. Cleaning

12

Caution

- | Only use drinking-quality water for cleaning.
- | Take apart and wash all parts that come in contact with the breast and breastmilk immediately after use to avoid dry up of milk residues and to prevent growth of bacteria.

Note

- | The pump set requires no maintenance.
- | Take care not to damage parts of the pump set during cleaning.
- | If the individual components of the pump set are cleaned in a dishwasher parts may be discoloured by food pigments. This has no impact on the function.

5.1 Before using for the first time and after every use

Disassemble the pump set into individual parts.

Rinse all parts with cold, clear water (approx. 20 °C) for 10–15 seconds.

2

Rinse all parts with cold, clear water (approx. 20 °C).

3

Clean all parts with plenty of warm soapy water (approx. 30 °C).

5

Dry with a clean cloth or leave to dry on a clean cloth.

or

As an alternative to steps 2, 3 and 4.

If you clean the individual parts of the pump set in the dishwasher, place them on the top rack or in the cutlery section.

5. Cleaning

14

Caution

- I Only use drinking-quality water for cleaning.
- I Take apart and wash all parts that come in contact with the breast and breastmilk immediately after use to avoid dry up of milk residues and to prevent growth of bacteria.

Note

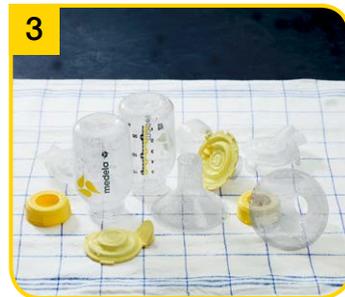
- I When boiling, a tea spoon of citric acid may be added to avoid a build-up of limescale.
- I Store the pump set in a clean bag/ container until next use. Or store it in clean paper or a cloth.

5.2 Before using for the first time and once a day

More information on page 12, chapter 5.1 steps 2–4.

Disassemble the pump set into individual parts.

Dry with a clean cloth or leave to dry on a clean cloth.

2a

Cover all parts with water and boil for five minutes.

or

2b

Use QuickClean* bags in the microwave in accordance with the instructions.

5.3 Cleaning the motor unit

1

Wipe off with a clean, damp cloth.

*More information at www.medela.com

6. Preparing for pumping

16

Warning

! re 6.1 Mains operation:

Follow the **exact** order of the steps.

I Only use the mains adapter supplied with the Mini Electric.

I Make sure the voltage of the mains adapter is compatible with the power source.

Info

! re 6.2 Battery operation:

The batteries do not need to be removed when the main adapter is used for pumping.

I Regularly check the condition of the batteries in the motor unit.

I If Mini Electric will remain unused for a longer period of time, remove the batteries from the battery compartment.

6.1 Mains operation

Plug the mains adapter cord into the motor unit.

6.2 Battery operation

Remove the battery compartment cover.

2

Plug the mains adapter into the mains socket.

2

Insert two AA LR6 batteries.

3

Slide the cover back.

6. Preparing for pumping

18

Caution

I Wash hands thoroughly (at least one minute) with soap and water before touching the breast and pump parts.

Note

- I Use Medela original accessories only.
- I Check pump set components for wear or damage before use and replace if necessary.
- I All components must be completely dry before use.

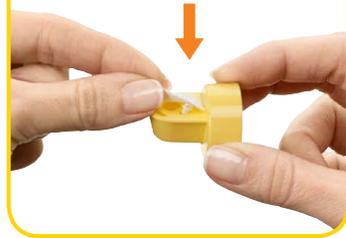
Info

I Carry out all the steps with care and assemble the pump set correctly. Otherwise a good vacuum may not be achieved.

6.3 Assembling the pump set

1

Carefully push the valve membrane onto the valve head until it engages.

4

Screw the bottle onto the breastshield.

2

Push the breastshield onto the connector.

3

Push the valve head and membrane onto the connector.
→ Position valve head **laterally**.

5

Insert the motor unit into the connector.

6

Rotate motor unit clockwise until the long battery compartment is located over the bottle.

7. Expression

20

Caution

1 re step 2 (chapter 7.1):

Answer the following questions to find out whether the available breastshield size will fit:

- | Is your nipple moving freely in the tunnel?
- | Is minimal or no areolar tissue being pulled into the tunnel of the breastshield?
- | Do you see a gentle, rhythmical motion in the breast with each cycle of the pump?
- | Do you feel the breast emptying all over?
- | Is your nipple pain-free?

If the answer to any of these questions is “No” or if you experience problems or pain during expression, consult your breastfeeding and lactation consultant or midwife.

Note

1 re step 2 (chapter 7.1):

Do not hold the pump set by the bottle. This can lead to blockage of the milk ducts and engorgement.

- | Wipe the breast with a warm washcloth (do not use alcohol).

7.1 Preparing for pumping

Place the breastshield on your breast so that your nipple is properly **centred** in the tunnel.

7.2 Pumping

Set the vacuum control to minimum.

2

Hold the breastshield onto your breast with your thumb and index finger. Support your breast with the palm of your hand.

2

Switch on the breastpump.
→ Slide the switch to Start.

3

Turn the vacuum control up towards MAX until it is slightly uncomfortable. Then turn down slightly towards MIN.
→ Make sure that the milk flows into the bottle properly.

7. Expression

22

Caution

I Contact your midwife or lactation consultant if you can express only minimal or no milk or if expression is painful.

Note

I Always disconnect the Mini Electric from the mains after expression.
I Only fill the bottle to the 150 ml mark.

4

Turn off breastpump.
→ Slide switch to Stop.

5

Use the stand to prevent the bottle from tipping over.

6

Close the bottle with a lid.
→ Follow the instructions in Chapter 8 “Storing and thawing breastmilk”.

Clean in accordance with chapter 5, page 12.

8. Storing and thawing breastmilk

24

8.1 Storage

Storage guidelines for freshly expressed breastmilk (for healthy terms babies)

Room temperature	Refrigerator	Freezer	Thawed breastmilk
4–6 hours at 19 to 26 °C (66 to 78 °F)	3–8 days at 4°C (39°F) or lower	6–12 months –18 to –20 °C (0 to 4 °F)	In refrigerator for 10 h max. Do not refreeze!

I Do not store breastmilk in the door of the refrigerator. Instead choose the coldest part of the refrigerator (this is at the back of the glass shelf above the vegetable compartment).

8.2 Freezing

I You can freeze expressed breastmilk in milk bottles or “Pump & Save”^{*} bags. Do not fill the bottles or bags more than 3/4 full to allow space for possible expansion.

I Label the bottles or “Pump & Save” bags with the date of expression.

8.3 Thawing

Caution

Do not thaw frozen breastmilk or do not warm breastmilk in a microwave or a pan of boiling water to avoid loss of vitamins, minerals and other important components and to prevent burns.

I In order to preserve the components of the breastmilk, thaw the milk in the refrigerator overnight. Alternatively, you can hold the bottle or “Pump & Save” bag under warm water (max. 37 °C).

I Gently swirl the bottle or “Pump & Save” bag to mix any fat that has separated. Avoid shaking or stirring the milk.

9. Feeding breastmilk

Info

Medela recommends Calma* for feeding breastmilk.

With Calma

| your baby can feed, pause and breathe regularly.

| milk will flow when a vacuum is created.

| your baby's natural feeding behaviour will be maintained, supporting easy transition back to the breast.

Calma – unique feeding solution for breastmilk.

You can find detailed information about Calma on the Medela website (www.medela.com).

10. Troubleshooting

26

Problem	Solution
If the motor is not running	Check presence of mains current. Check that the device is switched on.
If there is loss of power during battery operation	Replace batteries.
If there is low or no suction	Ensure the breastshield forms a complete seal around the breast. All the connections are secure. The valve membrane is lying flat against the valve head. Check that the valve membrane and valve head are clean and undamaged. Motor must be properly secured to the connector. All components must be completely dry.
If milk accidentally gets into the motor	Contact your Medela dealer or the store where you bought the pump.

11. Warranty / disposal

Warranty

The "International Warranty" brochure contains detailed warranty information.

Disposal

The unit is made of various metal and plastics. Before disposal, the device is to be rendered unusable and it must not be disposed of as unsorted municipal waste in accordance with local regulations. Use your local return and collection system for waste electrical and electronic equipment (incl. batteries). Improper disposal may have harmful effects on the environment and on public health.

12. Complete care during breastfeeding time

28

The following accessories for the Mini Electric are available from Medela distribution partners by specifying the product number.

Spare parts

Article number	Product
099.0032	Mini Electric Adapter (120V; 3V) US / JP
099.0033	Mini Electric Adapter (230V; 3V) Euro
099.0034	Mini Electric Adapter (240V; 3V) AUS / NZ
099.0035	Mini Electric Adapter (240V; 3V) AUS / UK
099.0030	Mini Electric Motor Unit

Accessories

Article number	Product
008.0337*	PersonalFit 2 Breastshields S (21 mm)
008.0338*	PersonalFit 2 Breastshields M (24 mm)
008.0339*	PersonalFit 2 Breastshields L (27 mm)
008.0340*	PersonalFit 2 Breastshields XL (30 mm)
008.0341*	PersonalFit 2 Breastshields XXL (36 mm)
008.0073	Breastmilk Bottle 150 ml (3 pcs)
008.0075	Breastmilk Bottle 250 ml (2 pcs)
008.0145	Calma Solitaire
200.1756	Cooling Bag
200.0625	CityStyle Breastpump Bag

Other Medela products are available on the www.medela.com web site.

* If the result of expression is unsatisfactory or expression is painful, contact your midwife or lactation consultant. A different size of PersonalFit™ Breastshield may help make expression more comfortable and successful.

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13. EMC/Technical description

The Mini Electric breastpump needs special precautions regarding EMC (Electromagnetic compatibility) and needs to be installed and put into service according to the EMC information provided in these instructions for use. Wireless communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, walkie-talkies can affect the electric breastpump Mini Electric and should be kept at least a distance 1.0 m away from the equipment. Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 1)

Electromagnetic emissions		
The electric breastpump Mini Electric is intended for use in the electromagnetic environment specified below. The customer or the user of the electric breastpump Mini Electric should assure that it is used in such an environment.		
Emission tests	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The electric breastpump Mini Electric uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Class A	The electric breastpump Mini Electric is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

 **Warning** – The electric breastpump Mini Electric should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the electric breastpump Mini Electric should be observed to verify normal operation in the configuration in which it will be used.

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 2)

Electromagnetic immunity			
<p>The electric breastpump Mini Electric is intended for use in the electromagnetic environment specified below. The customer or the user of the electric breastpump Mini Electric should assure that it is used in such an environment.</p>			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient / burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input / output lines	± 2 kV for power supply lines Not applicable ^a	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV differential mode Not applicable ^b	Mains power quality should be that of a typical commercial or hospital environment.
^a No input/output lines ^b No earth, class II equipment			

13. EMC/Technical description

<p>Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11</p>	<p><5 % U_T (>95 % dip in U_T) for 0,5 cycle 40 % U_T (60 % dip in U_T) for 5 cycles 70 % U_T (30 % dip in U_T) for 25 cycles <5 % U_T (>95 % dip in U_T) for 5 s</p>	<p><5 % U_T (>95 % dip in U_T) for 0,5 cycle 40 % U_T (60 % dip in U_T) for 5 cycles 70 % U_T (30 % dip in U_T) for 25 cycles <5 % U_T (>95 % dip in U_T) for 5 s</p>	<p>Mains power quality should be that of a typical commercial or hospital environment. If the user of the electric breastpump Mini Electric requires continued operation during power mains interruptions, it is recommended that the electric breastpump Mini Electric is powered from an uninterruptible power supply or a battery.</p>
<p>Power frequency (50/60 Hz) magnetic field IEC 61000-4-8</p>	<p>3 A/m</p>	<p>3 A/m</p>	<p>Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.</p>
<p>NOTE U_T is the a.c. mains voltage prior to application of the test level.</p>			

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 4)

Electromagnetic immunity			
<p>The electric breastpump Mini Electric is intended for use in the electromagnetic environment specified below. The customer or the user of the electric breastpump Mini Electric should assure that it is used in such an environment.</p>			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
<p>Conducted RF IEC 61000-4-6</p>	<p>3 Vrms 150 kHz to 80 MHz</p>	<p>3 Vrms</p>	<p>Portable and mobile RF communications equipment should be used no closer to any part of the electric breastpump Mini Electric, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance $d = 1,2\sqrt{P}$</p>
<p>Radiated RF IEC 61000-4-3</p>	<p>3 V/m 80 MHz to 2,5 GHz</p>	<p>3 V/m</p>	<p>$d = 1,2\sqrt{P}$ 80 MHz to 800 MHz $d = 2,3\sqrt{P}$ 800 MHz to 2,5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b Interference may occur in the vicinity of equipment marked with the following symbol:</p>

13. EMC/Technical description

Note 1 At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the electric breastpump Mini Electric is used exceeds the applicable RF compliance level above, the electric breastpump Mini Electric should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the electric breastpump Mini Electric.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 10 V/m.

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 6)

Recommended separation distances between portable and mobile RF communications equipment and the electric breastpump Mini Electric

The electric breastpump Mini Electric is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the electric breastpump Mini Electric can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the electric breastpump Mini Electric as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter M		
	150 kHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3\sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,37	0,37	0,74
1	1,2	1,2	2,3
10	3,7	3,7	7,4
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

Note 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

13. EMC/Technical description

Electromagnetic environments		
Environment	Locations	General characteristics
Typical health care	Hospital, large clinic, doctor's office	Partly controlled, covered by the general requirements of this collateral standard
Residential	Doctor's office, small clinic	Not controlled, health care professional present
Residential	Home	Not controlled, health care professional not normally present
Transport, mobile	Car, aircraft (fixed-wing and helicopter), ambulance	Not controlled, wide variations, critical receivers nearby, harsh environments for ESD, RF, electric and magnetic fields
Special	Operating theatre, emergency room	Case-by-case examination of environment

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Medela AG