*No. 1 choice
of hospitals
& mothers*

Lactina®

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Thank you for choosing the Lactina Breastpump. Breastmilk is the natural choice for your baby, but there are reasons and situations where breastfeeding is not possible. For over 50 years, it has been Medela's conviction that these mothers and babies can be supported. In close co-operation with breastfeeding experts, Medela, one of the market leaders, has researched and developed breast-pumps that perfectly meet breastfeeding mothers' needs. For mothers who insist on the best for their babies and themselves. Because you care.

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TR
116–143

Table of Contents

1. Intended use/Population – Contraindication	5
2. Meaning of symbols	6
3. Important safety information	8
4. Product description	10
5. Cleaning	12
5.1 Before using for the first time and after every use	12
5.2 Before using for the first time and once a day	14
5.3 Cleaning the breastpump and case	15
5.4 Disinfecting the breastpump	15
6. Preparing for pumping	16
6.1 Assembling the pump set	16
6. Preparing for pumping	18
7. Expression	20
7.1 Preparing for single pumping	20
7.3 Preparing for hands-free expression with the Easy Expression Bustier	22
7.4 Preparing for double pumping	24
7.5 Manual pumping	24
8. Storing and thawing breastmilk	26
8.1 Storage	26
8.2 Freezing	26
8.3 Thawing	26
9. Feeding breastmilk	27
10. Troubleshooting	28
11. Warranty and maintenance/disposal	29
12. Complete care during breastfeeding time	30
13. Technical specifications	144

1. Intended use/Population – Contraindication

5

The intended use and indications of breastpumps

Breastpumps are intended to be used by lactating women in the hospital or home environment for expressing breastmilk. Breastpumps are indicated to alleviate the symptoms of engorgement by expressing milk. In case of mastitis breastpumps support the healing process by removing breastmilk from the affected breast. Breastpumps are indicated to relieve sore and cracked nipples and to bring out flat or inverted nipples. Also breastpumps enable mothers to provide breastmilk for babies who cannot feed directly at the breast for a variety of reasons such as latch-on problems, cleft palate or premature babies.

Intended patient population/user

Adolescent paediatric or adult lactating women. Many women find it convenient, or even necessary, to use a breastpump to express and store their breastmilk once they have returned to work, are travelling or are otherwise separated from their baby. Women are often delighted to learn that a breastpump can be used as a supplement to breastfeeding and that some pumps are designed to mimic the suckling of a nursing baby.

Contraindications

There are no known contraindications for the Lactina breastpump.

2. Meaning of symbols

6

The warning symbol identifies all instructions that are important for safety. Failure to observe these instructions can lead to injury or damage to the breastpump. When used in conjunction with the following words, the warning symbol stands for:

Warning

Can lead to serious injury or death.

Caution

Can lead to minor injury.

Note

Can lead to material damage.

Info

Useful or important information that is not related to safety.

Symbols on the packaging

This symbol indicates that the material is part of a recovery/recycling process.

This symbol indicates a carton package.

This symbol indicates to keep the device away from sunlight.

This symbol indicates to handle the fragile device with care.

This symbol indicates the temperature limitation for operation, transport and storage.

This symbol indicates the humidity limitation for operation, transport and storage.

This symbol indicates the atmospheric pressure limitation for operation, transport and storage.

This symbol indicates to keep the device dry.

This symbol indicates that the packaging contains products intended to come in contact with food according to regulation 1935/2004.

This symbol indicates unique GSI Global Trade Item Numbers (GTIN).

Symbols on the device

This symbol indicates to follow instructions for use.

This symbol indicates the compliance with the essential requirements of the Council Directive 93/42/EEC of 14 June 1993 concerning medical devices.

This symbol indicates the manufacturer.

This symbol indicates do not dispose the device together with unsorted municipal waste (for EU only).

This symbol indicates the compliance with additional USA and Canada safety requirements for medical electrical equipment.

This symbol indicates a type BF applied part.

This symbol indicates manufacturer's catalogue number of the device.

IP21

This symbol indicates the protection against ingress of solid foreign objects and against harmful effects due to the ingress of water.

This symbol indicates the date of manufacture (four digits for the year and two digits for the month).

This symbol indicates that the Lactina is a class II device.

This symbol indicates fuse (to identify fuse boxes or their location).

3. Important safety information

8

Failure to follow these instructions/safety information can lead to danger from the device. Subject to technical changes.

Device

Danger of electrocution! Keep the device dry! Never immerse in water or other liquids!

The Lactina breastpump is not heat-resistant: keep away from radiators and open flames.

Do not expose the motor unit to direct sunlight.

There are no user-serviceable parts inside the Lactina.
Repairs must be performed only by an authorised service agency.
Do not repair yourself! No modifications to the device are permitted.

Never use a damaged device. Replace damaged or worn parts.

The operating life of the Lactina breastpump is 3000 hours.
The service life is seven years if used normally.

Electrical use

Separation from the mains is only assured through the disconnection of the power cord from the wall socket outlet.

Keep the power cord away from heated surfaces.

Do not reach for any electrical device if it has fallen into water.
Unplug immediately.

The Lactina breastpump should never be left unattended when plugged into a power source.

Use

Use the Lactina breastpump only for its intended use as described in this manual.

Never operate an electrical device if it has a damaged power cord, if it is not working properly, if it has been dropped or damaged, or dropped into water.

Do not use the Lactina breastpump while bathing or showering.

Do not use the Lactina breastpump while sleeping or overly drowsy.

Do not drive whilst hands-free pumping.

Supervision is necessary when the Lactina breastpump is used in the vicinity of children.

Contact your lactation consultant or physician if you experience problems or pain.

Radio frequency communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations and walkie-talkies can affect the Lactina breastpump. Due to electro-magnetic compatibility (EMC) a distance of at least 1 meter between the Lactina breastpump and the wireless equipment is recommended.

Important:

- I Plastic bottles and component parts become brittle when frozen and may break when dropped.
- I Also, bottles and component parts may become damaged if mishandled, e.g. dropped, over-tightened, or knocked over.
- I Take appropriate care in handling bottles and components.
- I Do not use the breastmilk if bottles or components become damaged.

4. Product description

10

Lactina reusable single or double pump set (applied parts)

The Lactina single or double pump set is an accessory for the Lactina breastpump. The pump set is suitable for expression by using the Lactina Select or Lactina Electric Plus. The pump set must only be used by one mother; use by more than one mother could pose a health risk.

The double pump set contains two pieces of each part except for the parts shown in yellow below.

The pump set contains:

The PersonalFit breastshield, size M (24mm). You can find more information on the correct choice of breastshield at www.medela.com/personalfit or scan the QR code on page 30.

**1x PersonalFit™
Breastshield M (24 mm)**
(for other breastshield sizes
see Chapter 12, page 30)
810.7084

1x valve head
810.0475

**1x breastmilk bottle,
150 ml transparent**
811.0020

**1x stand for
150 ml bottle**
810.0462

1x piston
810.0494

1x lid with opening
810.7062

1x disc
810.0473

1x protective cap
810.0474

**1x PersonalFit™
Connector**
810.7088

**1x silicon tubing
(105 cm)**
800.0828

**2x valve
membrane**
820.7013

**1x cylinder
holder**
810.0668

1x port cap
820.0135

**1x vacuum
regulator ring**
200.2775

1x cylinder
810.0717

1x rubber seal
200.2575

Rental pump required: Lactina Select or Lactina Electric Plus

Lactina Select
600.0929

Lactina Electric Plus
600.0930

Case
800.0809

Only Lactina Select

The Lactina Select has an additional knob for adjusting the pump speed between 40 and 60 cycles per minute. Select a fast pumping rhythm to stimulate the milk flow. Reduce the speed when the milk begins to flow.

Ideal accessories

Calma – the unique breastmilk feeding solution, the ideal way to feed your child expressed breastmilk. Calma is the result of extensive research and development.

Easy Expression Bustier makes hands-free pumping easier, especially with simultaneous expression from both breasts. It is available in 4 different sizes (S, M, L, XL).

More information on all Medela products can be found at www.medela.com

5. Cleaning

12

Caution

- | Only use drinking-quality water for cleaning.
- | Take apart and wash all parts that come in contact with breast and breastmilk immediately after use to avoid dry up of milk residues and to prevent growth of bacteria.

Note

- | The pump set requires no maintenance.
- | Take care not to damage parts of the pump set during cleaning.
- | If the individual parts of the pump set are cleaned in a dishwasher, parts may be discoloured by food pigments. This has no impact on the function.
- | Inspect the tubing after each pumping session for condensation and/or milk. Follow the instructions on page 28 if there is condensation or milk in the tubing.

* Medela recommends to clean all parts of the pump set (incl. piston/cylinder, etc.) before used for the first time as shown in Chapter 5.2.

5.1 Before using for the first time and after every use

1

Disassemble the pump set into individual parts (see diagram). The cylinder holder, piston/cylinder and tubing do not normally need to be washed.*

4

Rinse all parts with cold, clear water (approx. 20°C) for 10–15 seconds.

2

Rinse all parts with cold, clear water (approx. 20 °C).

3

Clean all parts with plenty of warm water and basic dish-washer agent (approx. 30 °C).

5

Dry with a clean cloth or leave to dry on a clean cloth.

or

6

As an alternative to steps 2, 3 and 4.

If you clean the individual parts of the pump set in the dish-washer, place them on the top rack or in the cutlery section.

5. Cleaning

14

Caution

- I Only use drinking-quality water for cleaning.
- I Take apart and wash all parts that come in contact with the breast and breastmilk immediately after use to avoid dry up of milk residues and to prevent growth of bacteria.
- I **for chapter 5.3 Cleaning the breastpump:**
Unplug the breastpump before cleaning.

Note

- I **for step 2a (Chapter 5.2):**
When boiling, a teaspoon of citric acid may be added to the water to avoid a build-up of limescale.
 - I Store the pump set in a clean bag/container until next use. Or store it in clean paper or a cloth.
 - I **for step 1 (Chapter 5.3):**
Unplug the breastpump before cleaning.
 - I Do not spray or pour liquid directly onto the pump.
- * Medela recommends to clean all parts of the pump set (incl. piston/cylinder, etc.) before used for the first time as shown in Chapter 5.2.

5.2 Before using for the first time and once a day

1

More information
on page 12, Chapter
5.1. Steps 2–4

Disassemble the pump set into individual parts.
The cylinder holder, piston/cylinder and tubing do not normally need to be washed.*

3

Dry with a clean cloth or leave to dry on a clean cloth.

2a

or

2b

Cover all parts with water and boil for five minutes.

Use QuickClean bags* in the microwave in accordance with the instructions.

5.3 Cleaning the breastpump and case

5.4 Disinfecting the breastpump

1

Wipe off the pump and case with a clean, damp (not wet) cloth. Water and basic dishwasher agent or non-abrasive detergent may be used.

1

Wipe off with a clean, damp (not wet) cloth. Alcohol based disinfecting agent may be used.

* Accessories available for order can be found on page 30.
For more information please refer to www.medela.com

6. Preparing for pumping

16

Caution

I Wash hands thoroughly (at least one minute) with soap and water before touching the breast, clean pump parts and clean pump set.

Note

I Use Medela original accessories only.
I Check pump set components for wear or damage before use.

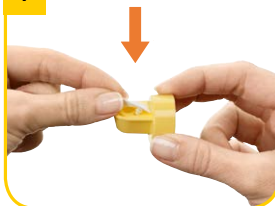
Info

I Carry out all the steps with care and assemble the pump set correctly. Otherwise a good vacuum may not be achieved.

6.1 Assembling the pump set

1

Carefully push the membrane onto the valve head until it engages.

4

Screw the bottle onto the connector.

2

Push the breastshield onto the connector.

3

Push the valve head and membrane onto the connector.
→ Position valve head **laterally**.

5

Insert the tubing into the corresponding opening in the connector.

6

Slide the piston into the cylinder.

6. Preparing for pumping

18

Warning

I for step 12 (chapter 6.1):

Use only the power cord that comes with Lactina.

I Make sure the voltage of the power cord is compatible with the power source.

7

Screw the piston/cylinder firmly into the cylinder holder.

10

Push the piston into the rubber clamp on the pump arm until it engages.

8

Insert the cylinder holder incl. piston/cylinder vertically into the opening on the front of the Lactina.

9

Rotate the cylinder holder until the pin sits in the slot on the Lactina.

19

11

Insert adapter end of tubing into opening in cylinder holder.

12

Plug the power cord into the mains socket.

7. Expression

20

Caution

I before step 2 (chapter 7.1):

Wipe the breast with a warm flannel (do not use alcohol).

I for step 2 (chapter 7.1):

Answer the following questions to find out whether the available **breastshield size** will fit:

I Is the nipple moving freely in the tunnel?

I Is minimal or no areolar tissue being pulled into the tunnel of the breastshield?

I Do you see a gentle, rhythmical motion in the breast with each cycle of the pump?

I Do you feel the breast emptying all over?

I Is your nipple pain-free?

If the answer to any of these questions is “No” or if you experience problems or pain during expression, consult your breastfeeding and lactation consultant or midwife.

I for step 3 (chapter 7.1):

Do not hold the pump set by the bottle. This can lead to blockage of the milk ducts and engorgement.

I for step 1 (chapter 7.2):

Select a fast pumping rhythm on the Lactina Select to start the milk flow. Reduce the speed when the milk begins to flow.

7.1 Preparing for single pumping

Insert the port cap into the unused opening of the cylinder holder.

Set the vacuum regulator ring to minimum.

The Lactina Select has an additional knob for adjusting the pumping speed.

2

Place the breastshield on your breast so that your nipple is properly **centred** in the tunnel.

3

Hold the breastshield onto your breast with your thumb and index finger. Support your breast with the palm of your hand.

7.2 Expression

5

Switch on the breastpump with .

1

Set vacuum comfort: twist down the vacuum regulator ring and increase the vacuum until it is slightly uncomfortable. Turn back to reduce the vacuum slightly.

7. Expression

22

Caution

I Contact your midwife or breastfeeding and lactation consultant if you can express only minimal or no milk or if expression is painful.

Note

I Always disconnect the Lactina from the power source after expression.

Info

I for chapter 7.3 and 7.4:

For pumping on both sides, place the port cap on the opening in the centre of the cylinder holder.

- I Do not kink the tubing during expression.
- I Double pumping saves time and increases the energy content of the milk. Milk production may be increased and maintained long-term.
- I Do not fill the bottle above max. volume indicator line.

2

Switch off the breastpump with

7.3 Preparing for hands-free expression with the Easy Expression Bustier*

1

Put on the Easy Expression Bustier and leave the zip open slightly. Take your assembled pump sets and take off the breastshields.

3

Use the stand or bottle holder to prevent the bottle from tipping over.

4

Clean in accordance with chapter 5, page 12.

Close the bottle with a lid or lid and disc (depending on bottle type).

→ Follow the instructions in chapter 8 "Storing and thawing breastmilk".

2

Place the breastshields on your breasts under the Bustier such that your nipples are properly **centred** in the tunnel of the breastshields.

3

More information on page 21, chapter 7.2 "Expression".

Close the zip and attach the assembled pump sets to the breastshields.

*The Easy Expression Bustier is available in 4 sizes. See page 30.
For more information please go to www.medela.com

7. Expression

24

Caution

! Do not hold the pump set by the bottle. This can lead to blockage of the milk ducts and engorgement.

Info

! Do not kink the tubing during expression.
! Double pumping saves time and increases the energy content of the milk. Milk production may be increased and maintained long-term.

! for step 2 (chapter 7.4):

The Lactina Select has an additional knob for adjusting the pumping speed. Select a fast pumping rhythm to stimulate the milk flow. Reduce the speed when the milk begins to flow.

7.4 Preparing for double pumping

1

More information on page 21, chapter 7.2 „Expression“.

Insert second adapter end of tubing into opening in cylinder holder. Place the breastshield on the breast as described in chapter 7.1, step 2 and 3.

7.5 Manual pumping

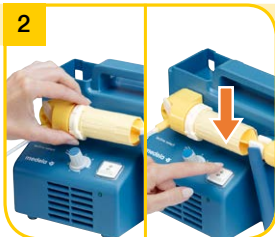
1

First follow steps 1–4, 6 on page 16, chapter 6.1.

Screw the cylinder onto the breastshield connector.

2

Set the vacuum regulator ring to minimum. Switch on the breastpump with .

3

Place the second breastshield on the breast so that the nipple is properly centred in the tunnel.

More on page 23,
chapter 7.2, step 3

2

Set the vacuum regulator ring to minimum and place the breastshield on the breast as in chapter 7.1 step 2 and 3.

3

Twist down the vacuum regulator ring and increase the vacuum until it is slightly uncomfortable. Turn back to reduce the vacuum slightly. Start with a manual pumping movement.

More on page 23,
chapter 7.2, step 3
et seq.

8. Storing and thawing breastmilk

26

8.1 Storage

Storage guidelines for freshly expressed breastmilk (For healthy term babies)

Room temperature	Refrigerator	Freezer	Thawed breastmilk
4–6 hours at 19 to 26 °C (66 to 78 °F)	3–8 days at 4 °C (39 °F) or lower	6–12 months –18 to –20 °C (0 to 4 °F)	In refrigerator for 10h max. Do not refreeze!

I Do not store breastmilk in the door of the refrigerator. Instead choose the coldest part of the refrigerator (this is at the back of the glass shelf above the vegetable compartment).

8.2 Freezing

- I You can freeze expressed breastmilk in milk bottles or “Pump & Save”^{*} bags. Do not fill the bottles or bags more than 3/4 full to allow space for possible expansion.
- I Label the bottles or “Pump & Save” bags with the date of expression.

8.3 Thawing

Caution

Do not thaw frozen breastmilk or do not warm breastmilk in a microwave or a pan of boiling water to avoid loss of vitamins, minerals and other important components and to prevent burns.

- I In order to preserve the components of the breastmilk, thaw the milk in the refrigerator overnight. Alternatively, you can hold the bottle or “Pump & Save” bag under warm water (max. 37 °C).
- I Gently swirl the bottle or “Pump & Save” bag to mix any fat that has separated. Avoid shaking or stirring the milk.

9. Feeding breastmilk

i Info

Medela recommends Calma for feeding breastmilk.

With Calma

I your baby can feed, pause and breathe regularly.

I milk will flow when a vacuum is created.

I your baby's natural feeding behaviour will be maintained supporting easy transition back to the breast.

Calma – unique feeding solution for breastmilk.

You can find detailed information about Calma on the Medela website (www.medela.com).

As a result of its comprehensive research work, Medela is one of the first companies in the world to incorporate babies' breastfeeding behaviour into its 2-Phase Expression and Calma innovations. 2-Phase Expression mimics the baby's natural sucking rhythm to express more milk in less time. Short, stimulating patterns are followed by longer ones, providing optimal milk flow. The unique Calma breastmilk feeding device allows the baby to pause and breathe during feeding whilst maintaining the sucking behaviour learned at the breast. Developed for a great breast-feeding experience.

10. Troubleshooting

28

Problem	Solution
If the motor is not running	Check whether mains power/battery is available. Check that the device is switched on.
If there is low or no suction	Assemble the pump set in accordance with the instructions in chapter 6. Express in accordance with chapter 7. Make sure that the breastshield forms a complete seal around the breast. Check whether all connections of the pump set and breastpump are secure. Express in accordance with chapter 7. Ensure the breastshield forms a complete seal around the breast. The white valve membrane must lie flat against the yellow valve head. Check that the valve head and membrane are clean and not damaged. See chapter 6.1 “Assembling the pump set”. I The ends of the tubing must be firmly inserted into the back of the breastshield and the opening on the cylinder holder. I The rubber seal is correctly secured to the piston. I The piston/cylinder is screwed onto the cylinder holder or breastshield correctly. I The cylinder holder including the piston/cylinder is fitted to the Lactina correctly. I During pumping on one side, the port cap has been inserted into the unused opening on the cylinder holder. I Do not kink or trap the tubing during expression. I All components must be completely dry.
If there is an overflow	1. Place the pump set into the bottle holder. 2. Turn off the pump and unplug the power cord from the socket. 3. Wipe off the pump with a damp (not wet) cloth. 4. If there is milk in the tubing: Remove the tubing and the piston/cylinder and clean according to the “Before using for the first time and once a day” instructions in chapter 5.2. Shake out water drops from the tubing and hang up the tubing to dry, or let the pump run with tubing attached for 1–2 minutes or until dry. If there is condensation in the tubing: Let the pump run with tubing attached for 1–2 minutes or until dry. 5. When components are completely dry, reassemble breastpump and accessories set.

11. Warranty and maintenance/disposal

Warranty and maintenance

3 years warranty on device from date of purchase excluding pump set and accessories, 6 months warranty on battery. The manufacturer is not liable for any damage or consequential damage caused by incorrect operation, inappropriate usage or use by unauthorized persons. Routine checks, servicing and battery changes may only be carried out by authorized Medela locations.

The engineering of the Medela Lactina Breastpump has been verified by independent test institutes for compliance with the EN / IEC 60601-1 standard. Copies of such certificates may be obtained on request.

Because of the method of construction used, Medela does not expect electrical safety to be affected at any time during the life of the product – provided that it is used properly in accordance with the intended use. There are no prescribed or recommended safety checks to be carried out.

Disposal

The unit is made of various metal and plastics. Before disposal, the device is to be rendered unusable and it must not be disposed of as unsorted municipal waste in accordance with local regulations. Use your local return and collection system for waste electrical and electronic equipment (incl. batteries). Improper disposal may have harmful effects on the environment and on public health. The accessories (pump set/bottles) are made of plastic and may only be disposed of in accordance with local regulations.

12. Complete care during breastfeeding time

30

The following accessories for the Lactina are available from Medela distribution partners by specifying the product name or number.

Article number	Product
008.0337*	PersonalFit Breastshield S (21 mm): EN/NL/FR
008.0338*	PersonalFit Breastshield M (24 mm): EN/NL/FR
008.0339*	PersonalFit Breastshield L (27 mm): EN/NL/FR
008.0340*	PersonalFit Breastshield XL (30 mm): EN/NL/FR
008.0341*	PersonalFit Breastshield XXL (36 mm): EN/NL/FR
008.0138	Calma Solitaire
008.0123	Calma with 150 ml milk bottle
008.0187	Calma with 250 ml milk bottle
008.0073	150 ml milk bottle (pack of 3): EN/RU/HU/CS/TR
008.0075	250 ml milk bottle (pack of 2)
008.0071	Pump & Save breastmilk bags EN/RU/HU/CS/TR
008.0217	Easy Expression Bustier white S
008.0220	Easy Expression Bustier white M
008.0224	Easy Expression Bustier white L
008.0225	Easy Expression Bustier white XL
008.0065	QuickClean bag

More information about all Medela products are available on our web site www.medela.com.

* If the result of expression is unsatisfactory or expression is painful, contact your lactation consultant or health care practitioner. A different size of PersonalFit Breastshield may help make expression more comfortable and successful.

Please note the information on breastshield sizes on page 20 (information column), visit www.medela.com/personalfit or scan the QR code.

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Medela AG